SVOX

SVOX STERİL İNHALASYON SUYU NEDİR ?

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WHAT IS THE SVOX STERILE INHALATION WATER?

- SVOX Sterile water for inhalation is a medical device used to provide humidification of the oxygen supplied to the patient.
- Designed as a bottle and contains deionised water (water for injection).
- SVOX Sterile Inhalation Water is water for injection and does not contain endotoxin.*
- The device must be used with an oxygen humidifier adapter.
- The device is for single use only.

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* According to the United States Pharmacopeia (USP), it is defined as water containing less than 0.25 IU/ml of endotoxin.

SVOX STERİL İNHALASYON SUYU SU KALİTESİ QUALITY OF SVOX STERILE INHALATION WATER

ISO 22519 Standartlarına göre önerilen su kalitesi ve SVOX inhalasyon suyu kalitesi.
According to ISO 22519 Standarts recommended water quality and SVOX inhalation water quality.

#	Parameter	RO Feed	After RO	PW	WFI**	SVOX
1	Hardness (PPN CaCO ₃)	≤feed water	<1	<1	<1	<0,014
2	TOC (ppb)	≤feed water	<500	<500 (online)	<500 (online)	<1
3	Endotoxin (EU / ml)	NA	NA	NA	<0.25	<0,25
4	Microbial total count (cfu / ml)	<500	<200	<100	< 10 cfu / 100 ml	<1
5	Free Chlorine (ppm)	<0.05	<0.05	<0.05	<0.05	0
6	Pseudomonas (cfu / 100ml)	<10	<1	<1	<1	0
7	E. coli (cfu / 100 ml)	<1	<1	<1	<1	0
8	Total coliforms, Fungus, (cfu / 100ml)	<1	<1	<1	<1	0
9	Conductivity (µS / cm@25 °C)	Like feed water	<10	<1.3 (online)	<1.3 (online)	<1

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** WFI (Water For Injecton) Column will be taken into consideration for comparison.

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WHAT IS SVOX STERILE INHALATION WATER USED FOR?

- Sterile inhalation water devices reduce dryness and irritation in the respiratory tract by humidifying the oxygen that patients inhale through the respiratory tract. This helps the patient to breathe comfortably.
- It offers comfort and relaxation to the patient by eliminating the dryness in oxygen with the bubble system.
- It is used for therapeutic purposes by bubble humidification with low oxygen flow.
- SVOX Sterile Inhalation Water does not contain any medication. It is only used to humidify oxygen with the bubble system.
- Patients who breathe by mouth, have chronic diseases, use nasal cannulas, have nose bleeds due to dryness and need oxygen support can all benefit from humidification during oxygen therapy.
- It can be used in patients of all ages and sexes who require oxygen administration and who are evaluated by the physician.

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PRODUCT COMPONENTS

- Inhalation Water Bottle
- Humidifier Adapter
- Inhalation Water; Deionised Water (Water for Injection)

STORAGE

- It should be stored in a dry and moisture-free environment at a temperature of 5 - 35 °C .

CERTIFICATION

- CE , MDR 2017 / 745,
- United States Pharmacopoeia (USP)

SHELF LIFE

- 24 months (2 years)

PACKAGING

- Humidifier adapters sterilised with ethylene oxide sterilisation, inhalation water bottle and user manual are packaged as a set in a transparent bag.

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Ref No	Product Name	Product Volume
-----	SVOX Sterile Inhalation Water	500 ml
-----	SVOX Sterile Inhalation Water	650 ml

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Gürbüz Sağlık Ürünleri Ltd. Şti.